



# TypeSafe® Segment Sampling Device


- 1** Insert the narrow end of the TypeSafe device into the test tube.
- Be sure the test tube is properly aligned in the test tube rack
  - Wear gloves when using TypeSafe

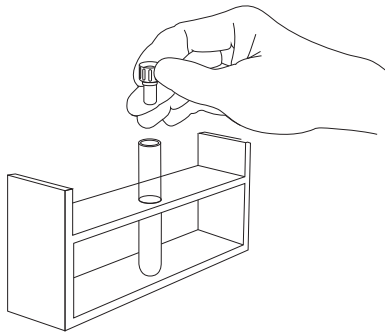

- 2** While holding the test tube and TypeSafe device steady with one hand, insert the blood segment into TypeSafe with the other hand, targeting the center of the device. Gently press until the needle has pierced the segment. Apply pressure to the segment, forcing blood into the test tube.
- Do not squeeze until the needle has pierced the segment
  - Be sure TypeSafe sits directly on top of the test tube during use, so the narrow end is facing straight down, reducing angles and gaps

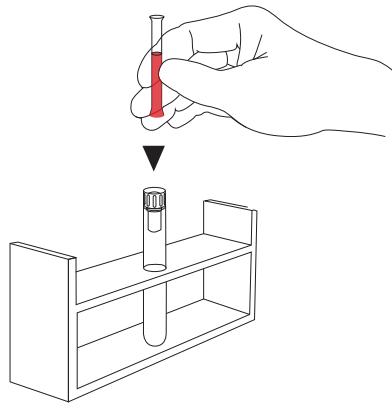

- 3** When the desired sample has been dispensed into the test tube, remove the blood segment and the TypeSafe device. Properly dispose of both items in a designated hazardous medical waste container.
- Single-use only
  - For in-vitro diagnostic purposes

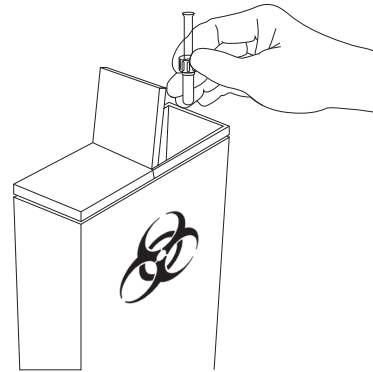

Visit [www.typhenex.com](http://www.typhenex.com) or call us at 866 897-3639 to learn more.

Typhenex Medical, LLC  
303 E. Wacker Drive, Suite 1200  
Chicago, IL 60601

AN 0703-008.3

