

# LipoClear<sup>®</sup>

## 95 Second lipemic sample clearing reagent

### Safe-Effective Process


- Inexpensive and convenient
- Non toxic, non-carcinogenic reagent, assures safe handling
- Most common analytes are not affected including enzymes and bilirubin

### Available in convenient prefilled tubes

- Low cost reagent is pre-dispensed into microtubes, no measuring required
- Two sample sizes available



The StatSpin high speed centrifuge is a perfect system for clearing lipemic samples in 95 seconds.

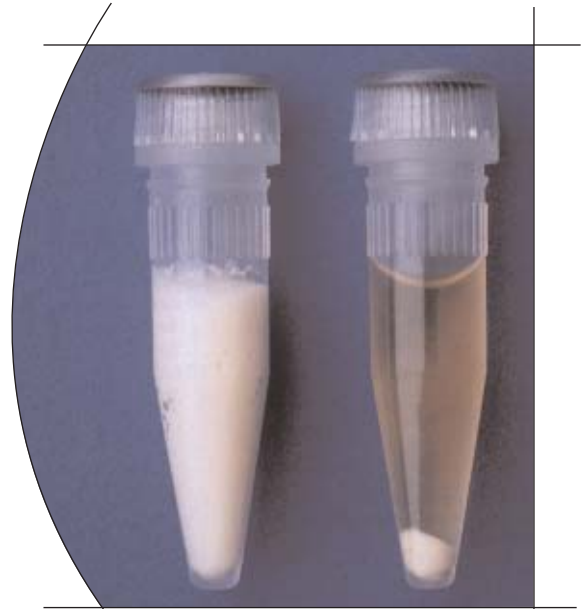

LipoClear reagent clears quickly and conveniently.

### Simple Procedure

- Add lipemic sample according to recommended procedure to reagent prefilled tube
- Mix well and let stand for 5 minutes
- Centrifuge and remove clarified sample for analysis
- Multiply results by 1.2



## StatSpin® Centrifuge

- Clears plasma/serum in 95 seconds, spinning at 15,800 rpm.
- Easy to load - rotor loads with simple push-on/pull-off action. No tools required.
- Easy to use - present time and speed. Light indicates cycle in process, beep signals completion.
- Whisper quiet - barely audible at 55dB.
- Small and lightweight.

|          |         |          |
|----------|---------|----------|
| Diameter | 6-5/8"  | 16.25 cm |
| Height   | 5-1/4"  | 13.12 cm |
| Weight   | 5.5 lbs | 2.5 kg   |

### Product #

SSMP

### Description

StatSpin® MP  
(100-240 V)

## Convenient low cost method

LipoClear reagent tubes and the StatSpin high speed centrifuge make a perfect system for clearing lipemic samples in only 95 seconds (after 5 minute mix and stand time).

## LipoClear Ordering Information

| Product # | Description                             |
|-----------|---|
| LC40      | 40 tubes per package for 0.5 mL samples |
| LC15      | 40 tubes per package for 1.5 mL samples |
| LC10      | 10 tubes per package for 0.5 mL samples |

### References:

"Evaluation of LipoClear," *Abstract presented at AACC 1989*, Linda C Rogers, Children's Memorial Hospital.



60 Glacier Drive  
Westwood, MA 02060  
Tel) 781.551.0100  
Fax) 781.551.0036  
800-782-8774  
www.proiris.com